



 allurion

Welcome to the

ElipseTM

Programme


No surgery
No endoscopy*
No anaesthesia
Just results**

*In rare cases, the Elipse Balloon may require endoscopic or surgical intervention for removal.


**Results may vary from one patient to another. A small percentage of patients don't respond to the treatment

This material is not intended for a U.S. audience. This material describes our product, which has not been approved or cleared for marketing by the U.S. Food and Drug Administration. Our product is not available for sale in the United States.

Before we get started...

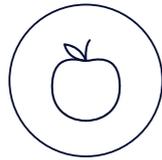


- 01 Which weight loss solutions have you previously tried?
- 02 What weight loss goal would you like to achieve with the Elipse programme?

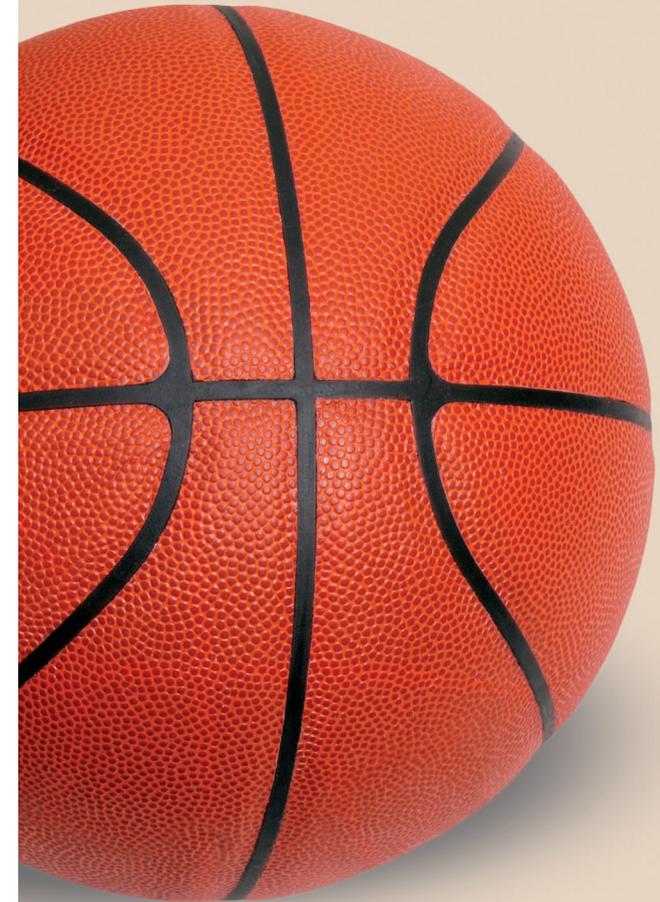

Not alone

Overweight and obesity
A global epidemic

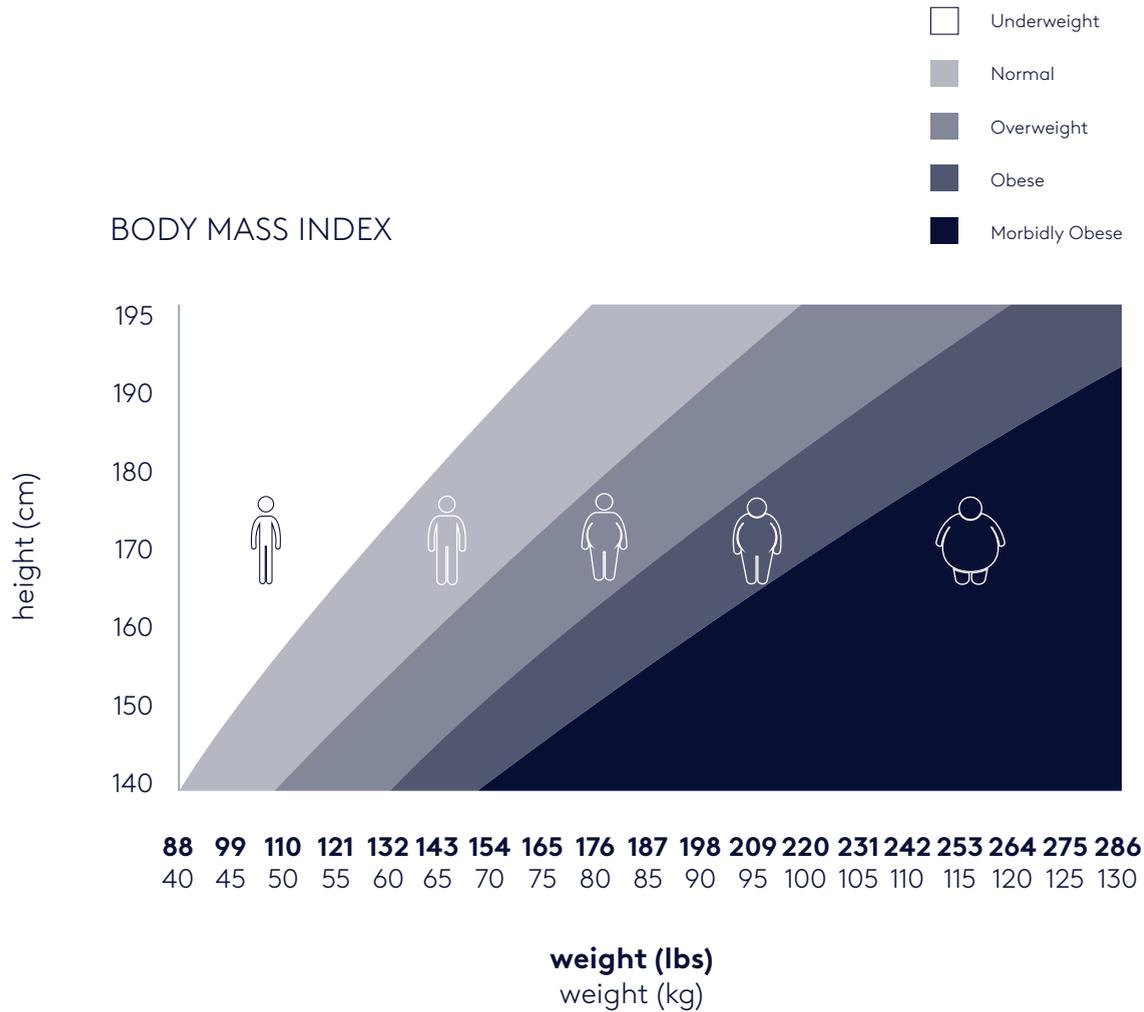

1 in 3


3

people are overweight or obese in France, Italy, Germany, Spain, the United Kingdom, China and the U.A.E*

*Based on WHO, Eurostat and US Census data


Effects of obesity are multidimensional

Excess weight impacts your health, not just the way you look and feel

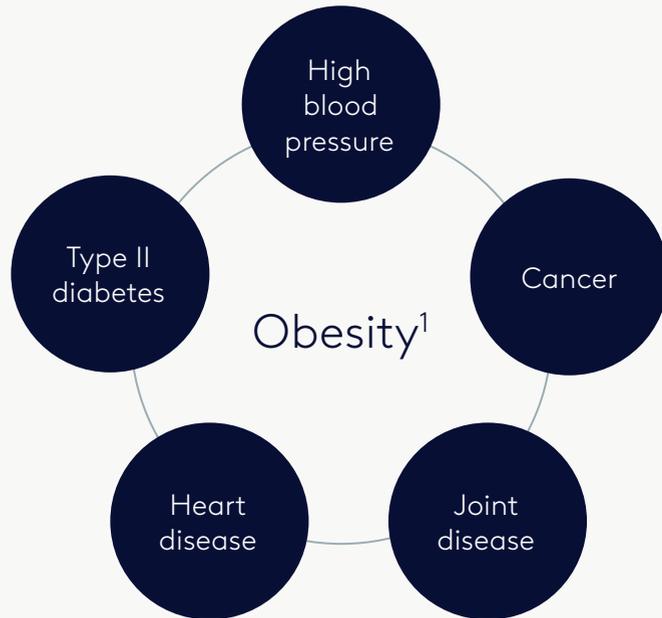

1. The health effects of overweight and obesity <https://www.cdc.gov/healthyweight/effects/index.html>. Last accessed 7 April 2019

MEDICAL COMPLICATIONS OF OBESITY

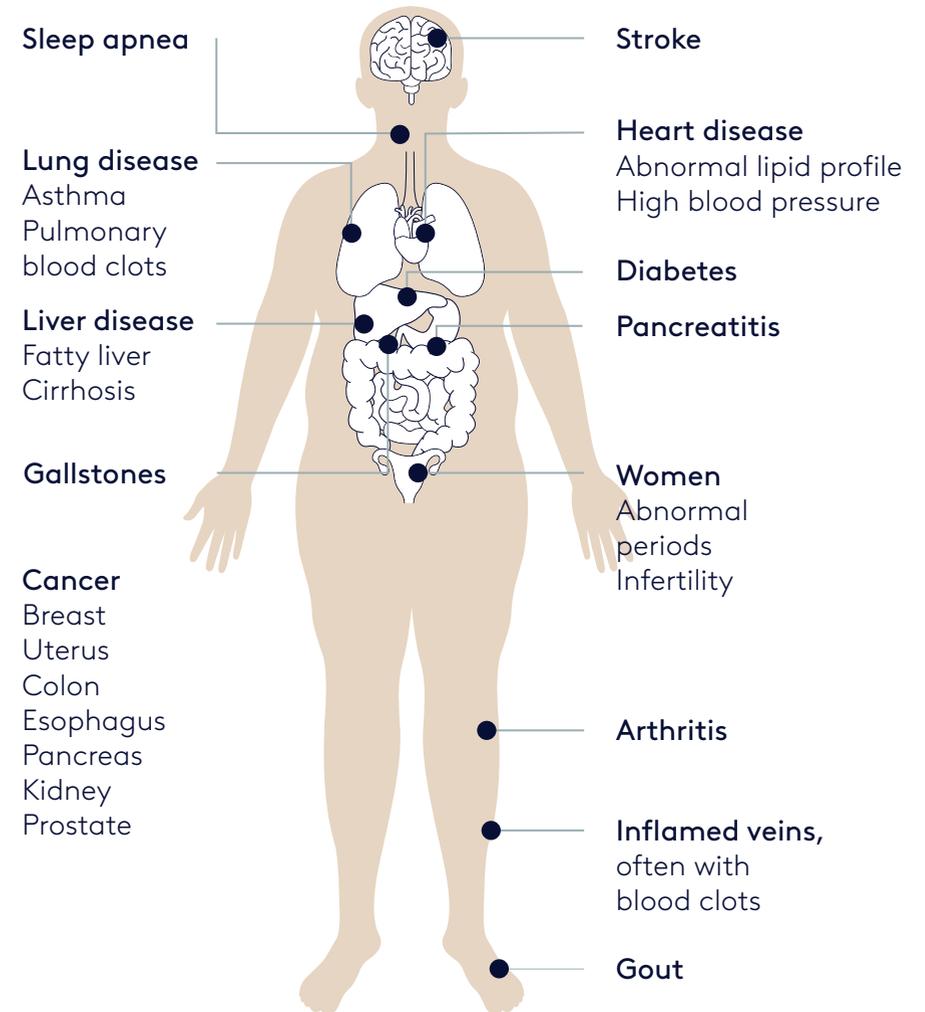


Source: Adapted from Yale University Rudd Center for Food Policy and Obesity

Frustrated with weight-loss programs?

According to Allurion market research, most of the available treatment options are not delivering on expectations

SATISFACTION WITH WL SOLUTIONS
Rated 1-10 scale - Of those that have used

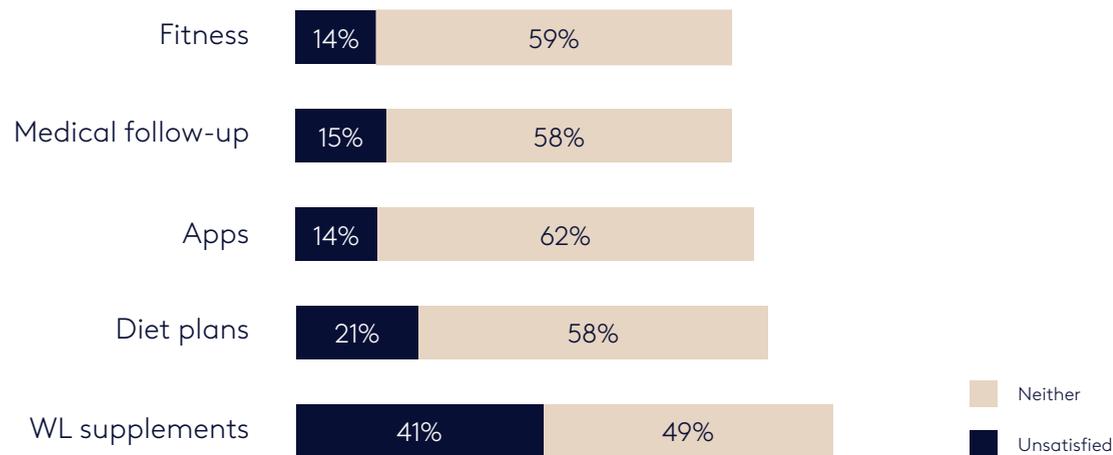

Q10. What would you be looking for from a physician / health care provider administered weight loss treatment? Global data. Base: n=2461. Likely to ask HCP for solution not tried.



76%

are not fully satisfied with previously tried weight loss solutions

Allurion consumer market research; 9,800 respondents; 8 geographies; Oct 2018
Q3C. On a scale of 1 to 10, where 1 is not at all satisfied and 10 is extremely satisfied, how satisfied are you with the following? Think about your experience of the whole process.
Global level. Base: n=4831
Satisfied: 8-10/10, neither 4-7/10, unsatisfied 1-3/10

Introducing the Elipse Programme

A fully supported programme designed for you



No
surgery

No
endoscopy*

No
anaesthesia

*In rare cases, the Elipse Balloon may require endoscopy or surgery for removal.

What ?

The Elipse Weight Loss Programme,
has demonstrated efficacy and the convenience
of a walk-in placement



After approximately
16 weeks of Elipse
therapy, patients lose
an average of **10 to 15kg**
(1.57 to 2.36 stone) and
10 to 15% of
body weight^{1 to 10}

1. Jamal et al. Obes Surg. 2019 Apr;29(4):1236-1241.
2. Raftopoulos and Giannakou. SOARD. 2017 Jul;13(7):1174-1182.
3. Al Sabah, S., et al. SOARD. 2018, 14(3), 311-317.
4. Machytka E et al. Endoscopy. 2017 Feb;49(2):154-160.
5. Genco et al. Obes Surg 2017,DOI 10.1007/s11695-017-2877-1
6. Al-Subaie S et al. Int J Surg. 2017 Dec;48:16-22.



Get multidisciplinary
support during your
weight loss journey^{*}

7. Ienca et al. Submitted to SOARD. Abstract A278, presented at Obesity Week, 2018, Nashville, USA
8. Genco et al. Submitted to SOARD. Abstract A185, presented at Obesity Week, 2018, Nashville, USA
9. Ienca et al. Abstract 237, presented at IFSO 2019, Madrid, Spain
10. Raftopoulos et al. Abstract 1978. Presented at ACS 2019, San Francisco, United States.




Discover a break-through
in simplicity: no surgery,
endoscopy or anaesthesia^{**}

^{*}The Elipse Balloon must be used in conjunction with a supervised nutrition and behaviour modification programme. Allurion advises a 6-month follow-up programme. Its duration is defined by the physician, and may vary depending on the physician and patient profile.
^{**}In rare cases, the Elipse Balloon may require endoscopic or surgical intervention for removal.

How?

Gastric Balloons induce weight loss by increasing satiety (fullness), delaying gastric emptying and reducing the amount of food eaten at each meal¹

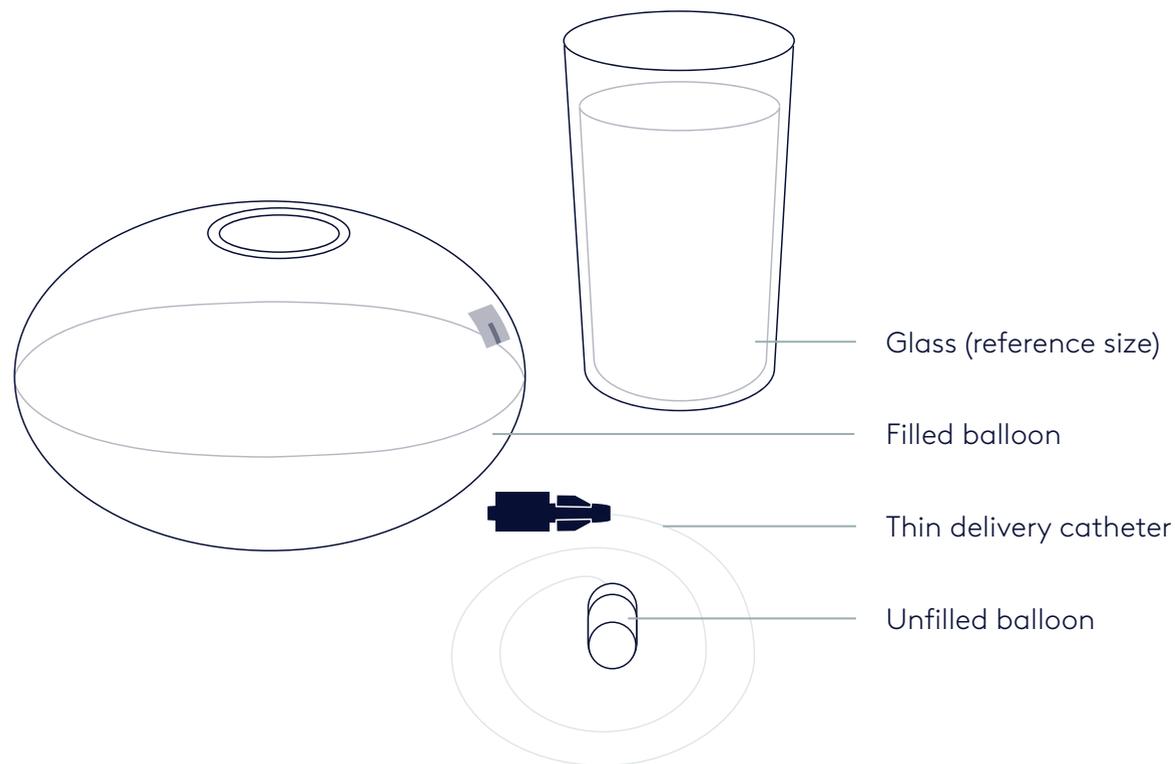


20 min

The Elipse Balloon experience takes place during a brief 20-minute walk-in placement²

1. Jamal et al. Obes Surg. 2019 Apr;29(4):1236-1241
2. Clinical Trial Reports: DAA069 and DAA105

How?

Gastric Balloons provide “a honeymoon from hunger”¹

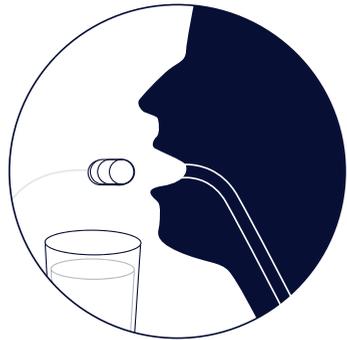


01

You swallow a capsule that contains the balloon attached to a thin tube

You will have an X-ray to make sure the capsule is in the right position

1. Quote from Lucy Jones, dietician <http://www.lucyjonesdietitian.co.uk/>

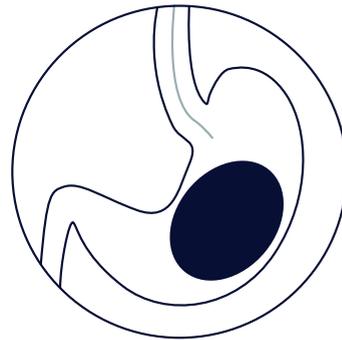


02

Once in your stomach, your doctor will use the tube to fill the balloon with liquid

You will have a second X-ray to confirm the balloon is filled. The doctor will then gently remove the tube and you'll be on your way

With the balloon in your stomach, you can eat less without feeling hungry

2. Jamal et al. *Obes Surg.* 2019 Apr;29(4):1236-1241

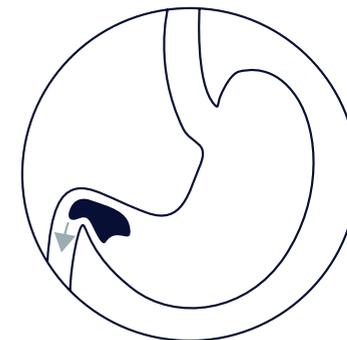


03

At approximately 16 weeks after placement, the Elipse Balloon self-empties and passes naturally^{2*}

*In rare cases, the Elipse Balloon may require endoscopic or surgical intervention for removal. Talk to your doctor today to find out if the Elipse Programme is right for you. You can also learn more at www.allurion.com.

How does the Elipse Balloon work?

The balloon takes up space in your stomach and induces weight loss by increasing satiety (fullness), delaying gastric emptying and reducing the amount of food eaten at each meal¹

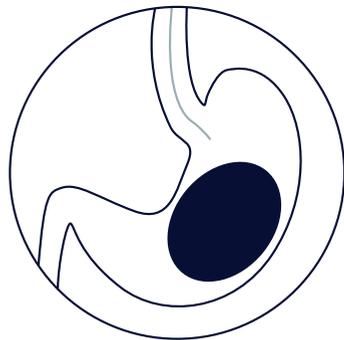

- Feel satisfied sooner → Smaller portions
- Creates a feeling of fullness → May reduce cravings
- Food stays in your stomach longer → May reduce snacking

1. Jamal et al. Obes Surg. 2019 Apr;29(4):1236-1241

How much weight can you expect to lose?



16 weeks

After approximately **16 weeks** of Eclipse therapy, patients lose an average of **10 to 15kg (1.57 to 2.36 stone)**^{1 to 10}


1. Jamal et al. Obes Surg. 2019 Apr;29(4):1236-1241.
2. Raftopoulos and Giannakou. SOARD. 2017 Jul;13(7):1174-1182.
3. Al Sabah, S., et al. SOARD. 2018, 14(3), 311-317.
4. Machytka E et al. Endoscopy. 2017 Feb;49(2):154-160.
5. Genco et al. Obes Surg 2017, DOI 10.1007/s11695-017-2877-1
6. Al-Subaie S et al. Int J Surg. 2017 Dec;48:16-22.



-15%

or around **10 to 15%** of their total bodyweight^{1 to 10}

7. Ienca et al. Submitted to SOARD. Abstract A278, presented at Obesity Week, 2018, Nashville, USA
8. Genco et al. Submitted to SOARD. Abstract A185, presented at Obesity Week, 2018, Nashville, USA
9. Ienca et al. Abstract 237, presented at IFSO 2019, Madrid, Spain
10. Raftopoulos et al. Abstract 1978. Presented at ACS 2019, San Francisco, United States.

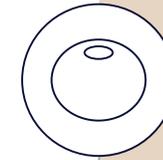

40k+

Eclipse Balloons have already been distributed* to help patients across Europe and the Middle East reach a healthier weight**

*As of January 2021

**Results may vary from one patient to another. A small percentage of patients don't respond to the treatment


A temporary balloon does not mean temporary weight loss

Lose Weight and Keep It Off

95%

of average weight loss was sustained at


12 month follow-up

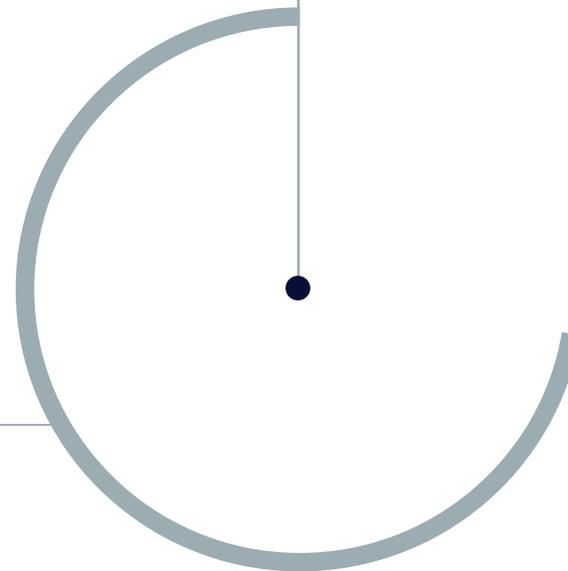


One clinical study with 509 patients has shown that 95% of average weight loss with Elipse can be sustained at 12 month follow up¹

Lifestyle modification

When coupled with a 16-week intensive lifestyle modification, Elipse leads to significant weight loss vs. lifestyle modification alone^{1*}

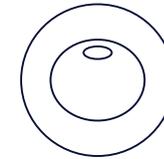

PERCENTAGE OF PATIENTS ACHIEVING 10% TBWL AT 16 WEEKS

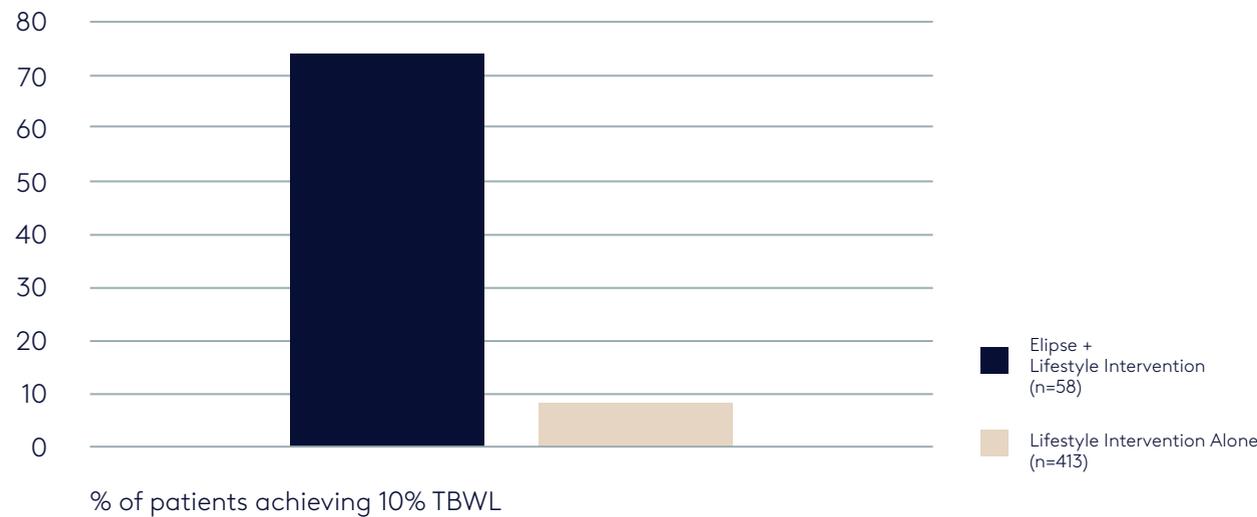

Intensive lifestyle modification included a structured curriculum-based nutritional, exercise and behaviour modification programme

1. Raftopoulos et al. Abstract 1978. Presented at ACS 2019, San Francisco, United States.

*The Elipse Balloon must be used in conjunction with a supervised nutrition and behaviour modification programme. Allurion advises a 6-month follow-up programme. Its duration is defined by the physician, and may vary depending on the physician and patient's profile.

Expert support

The Elipse Balloon is at the centre of a multidisciplinary weight loss programme



Support from your physician

6 months of dietary coaching from a trained nutritionist*

*The Elipse Balloon must be used in conjunction with a supervised nutrition and behaviour modification programme. Allurion advises a 6-month follow-up programme. Its duration is defined by the physician, and may vary depending on the physician and patient's profile.



Body composition scale and health tracker linked to your app to track your progress

The health tracker can be used to measure steps, exercise and distance



Use your app to track your weight loss progress, share with your healthcare team and loved ones

The app connects to your scales, Fitbit**, Apple Health**, Google Health** and the Allurion health tracker to track your daily activity

**Fitbit, Apple Health and Google Health are not owned by Allurion Technologies

Ten steps to success

Elipse is a fully supported lifestyle management programme

6 month plan**



*Results may vary from one participant to another. A small percentage of patients don't respond to the treatment
 **The Elipse Balloon must be used in conjunction with a supervised nutrition and behaviour modification programme. Allurion advises a 6-month follow-up programme. Its duration is defined by the physician, and may vary depending on the physician and patient profile.

1. Jamal et al. Obes Surg. 2019 Apr;29(4):1236-1241
2. Raftopoulos and Giannakou. SOARD. 2017 Jul;13(7):1174-1182.
3. Al Sabah, S., et al. SOARD. 2018, 14(3), 311-317.
4. Machytka E et al. Endoscopy. 2017 Feb;49(2):154-160.
5. Genco et al. Obes Surg 2017,DOI 10.1007/s11695-017-2877-1
6. Al-Subaie S et al. Int J Surg. 2017 Dec;48:16-22.

7. Ienca et al. Submitted to SOARD. Abstract A278, presented at Obesity Week, 2018, Nashville, USA
8. Genco et al. Submitted to SOARD. Abstract A185, presented at Obesity Week, 2018, Nashville, USA
9. Ienca et al. Abstract 237, presented at IFSO 2019, Madrid, Spain
10. Raftopoulos et al. Abstract 1978. Presented at ACS 2019, San Francisco, United States.

Continue to lose weight beyond 6 months

with a second Elipse balloon

1 year program



Prepare for the balloon

Regular follow-up with your dietician and learn healthy habits

01

1st balloon



The balloon helps you take a honeymoon from hunger and start your weight loss journey

Average

Max

-14.4 % body weight loss
-14.7 kgs weight loss

-22.3 % body weight loss

16

Maintain your healthy habits and continue dietician advice

02

Break

+1.4 kgs

Regular follow-up with your dietician and engage in healthier habits

03

2nd balloon*



The balloon helps you achieve your weight loss goal

-10.0 % body weight loss
-8.8 kgs weight loss

-20.9% body weight loss



Say hello to a new you!

Total (2 sequential balloons)

-22.8 % body weight loss
-22 kgs weight loss

-40.9% body weight loss
-48.7 kgs weight loss

* For subsequent balloons it is critical the patient still meets the requirements of the IFU

lenca et al. Accepted presentation at Obesity Week 2020

Best-In-Class Warranty

Have confidence in the Allurion Programme

We designed our warranty to give you maximum peace of mind and satisfaction when you choose the Allurion Programme. The Allurion Assurance Plus warranty is free of charge for all patients.

What is covered?

The balloon will last at least 90 days before it passes naturally, or will be replaced free of charge.

Up to \$8000 for out-of-pocket financial assistance for rare events of pancreatitis or hyperinflation that require endoscopic removal.

Easy review process and out-of-pocket financial assistance for rare events that require surgery

Full terms and conditions can be found at www.allurion.com/warranty




The Eclipse Programme


A complete weight loss package



Stay motivated
by monitoring your
progress each week

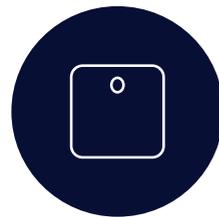

Use your stylish wireless
scale and easy-to-use
app to record real-time
data: BMI, weight, body
fat and more

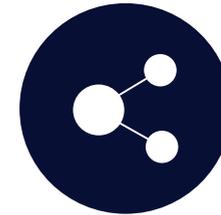

Send your progress
to your doctor, family
and friends

What do patients ask?

**Do you have any
specific questions
at this point?**

How widely is the Elipse Balloon used?

- The Elipse Balloon has been thoroughly tested and multiple clinical studies demonstrate that it is safe^{1 to 10}
- It has been used in the European Union since 2015




100+

It is used in more than 100 weight-loss centers across Europe and the Middle East*

*As of January 2021

**In rare cases, the Elipse Balloon may require endoscopic or surgical intervention for removal.

1. Jamal et al. *Obes Surg*. 2019 Apr;29(4):1236-1241
2. Raftopoulos and Giannakou. *SOARD*. 2017 Jul;13(7):1174-1182.
3. Al Sabah, S., et al. *SOARD*. 2018, 14(3), 311-317.
4. Machytka E et al. *Endoscopy*. 2017 Feb;49(2):154-160.
5. Genco et al. *Obes Surg* 2017, DOI 10.1007/s11695-017-2877-1
6. Al-Subaie S et al. *Int J Surg*. 2017 Dec;48:16-22.

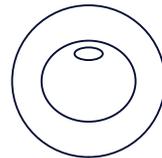


40k+

balloons have been distributed worldwide*

Unlike other gastric balloons, Elipse is the only gastric balloon requiring no invasive medical procedure**

7. Ienca et al. Submitted to SOARD. Abstract A278, presented at Obesity Week, 2018, Nashville, USA
8. Genco et al. Submitted to SOARD. Abstract A185, presented at Obesity Week, 2018, Nashville, USA
9. Ienca et al. Abstract 237, presented at IFSO 2019, Madrid, Spain
10. Raftopoulos et al. Abstract 1978. Presented at ACS 2019, San Francisco, United States.

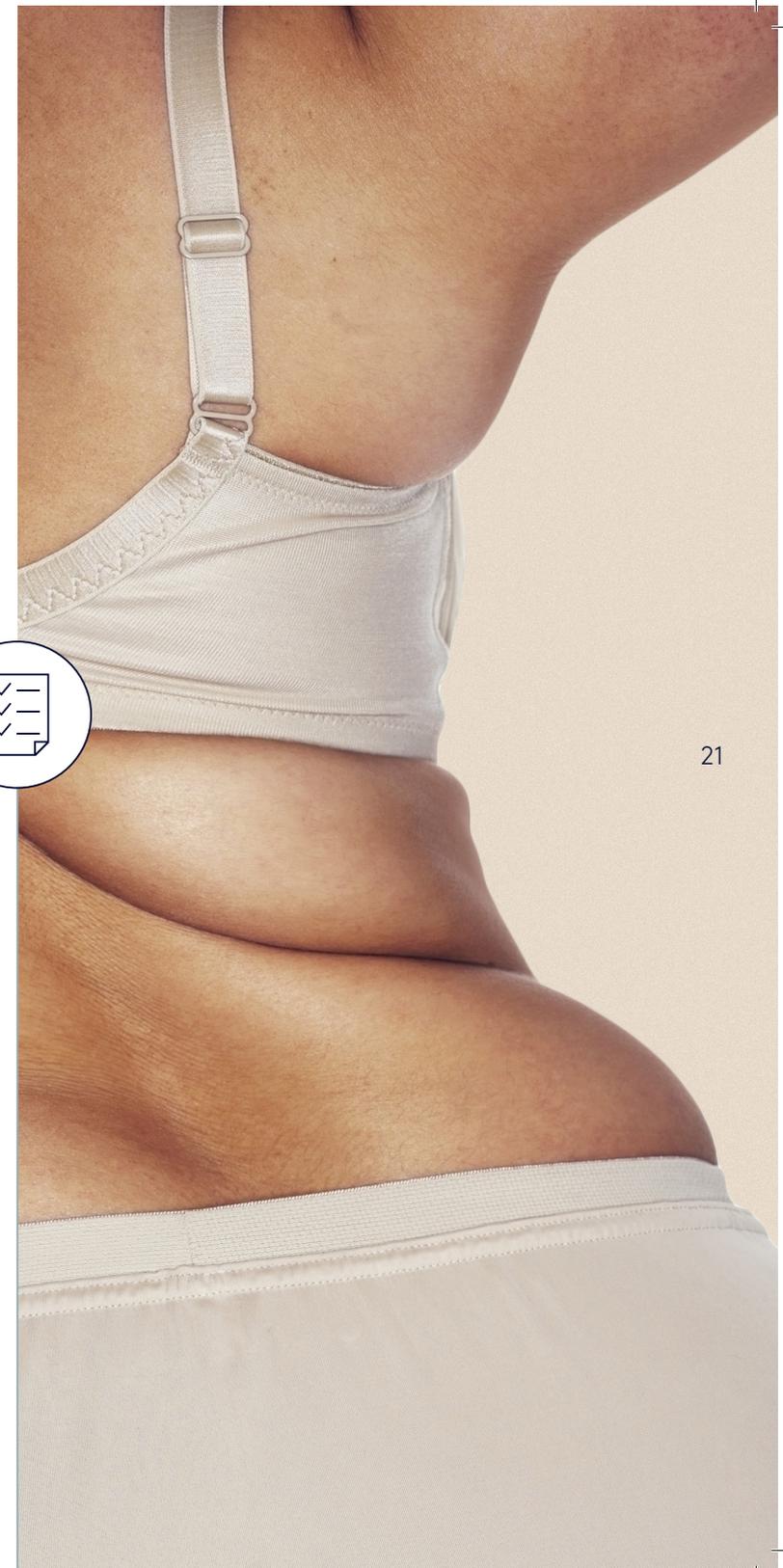


Eclipse Safety Profile

Allurion Internal Vigilance Data

≈ 98%

No reported adverse events

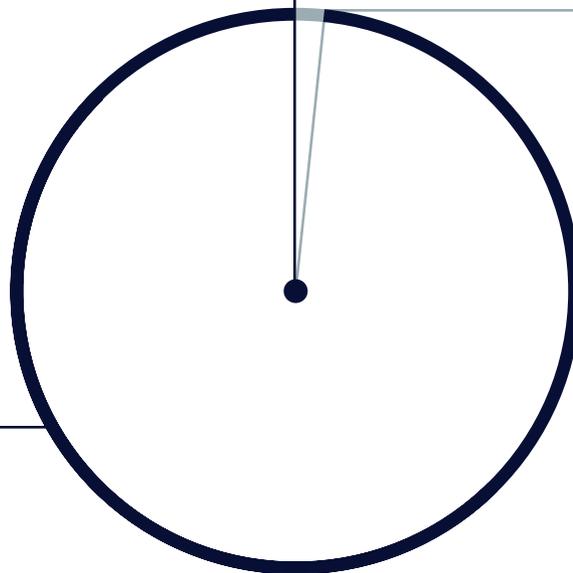

ADVERSE EVENTS RATES (%)
MARCH 2016 - JUNE 2019

Total units distributed= 15,200

Total adverse events reported were less than 2% of total Eclipse Balloons placed

The 10 Most Relevant Types

- Premature Deflation 0.43%
- Intolerance 0.25%
- Small Bowel Obstruction 0.11%
- Hyperinflation 0.05%
- Dysphagia 0.05%
- Gastric Outlet Obstruction 0.03%
- Gastric Perforation 0.02%
- Incomplete Device Emptying 0.02%
- Pancreatitis 0.01%
- Gastroparesis 0.01%


In rare cases, the empty balloon may be vomited instead of passing through the GI tract. Although startling to the patient, vomiting of the balloon has not been associated with any reported adverse events to date.

In rare cases, the Eclipse Balloon may require endoscopic or surgical intervention for removal. Some individuals may not be eligible to receive the Eclipse Balloon. Doctors take a detailed medical history and perform a physical examination to determine patients' eligibility.

The Eclipse Balloon is a temporary weight loss aid for adults over the age of 18 who are overweight or obese, defined as a body mass index (BMI) ≥ 27 . The Eclipse Balloon can only be administered by a healthcare professional who is knowledgeable about the use of the Eclipse Balloon.

The Eclipse Balloon must be used in conjunction with a supervised nutrition and behaviour modification programme. Allurion advises a 6-month follow-up programme. Its duration is defined by the physician, and may vary depending on the physician and patient's profile.

What if I can't swallow the capsule?

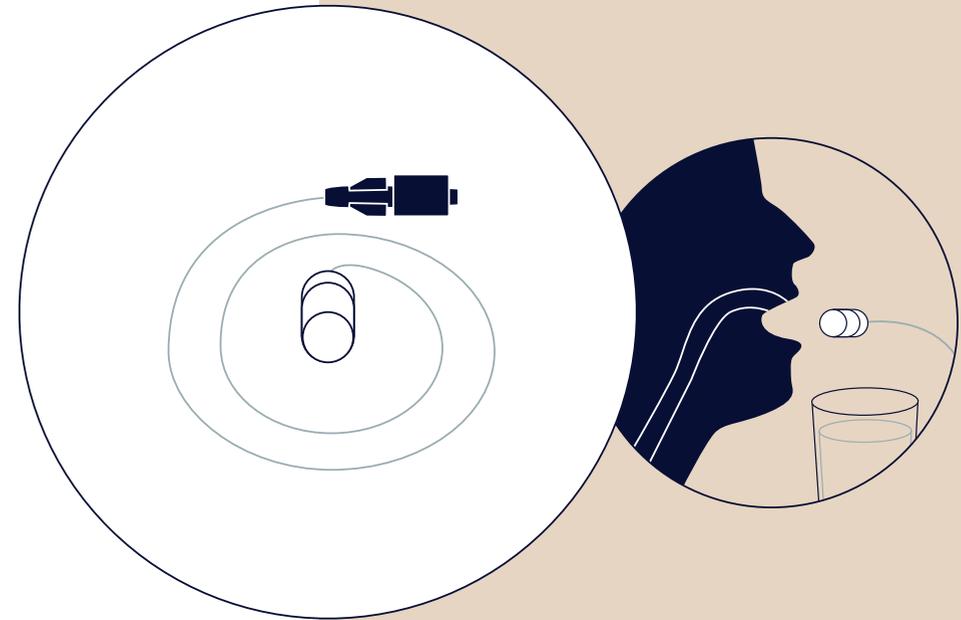

- Most patients are able to swallow the capsule¹
- There is the possibility for assisted swallowing:
- If you have trouble swallowing the capsule on your own, your physician can assist you by gently stiffening the catheter with a thin wire

In a recent clinical study of 1623 patients²

99.9% of patients were able to swallow the device with only 32% needing stylet assistance

1. Internal Allurion report: DAA069
2. Ienca et al. Abstract 237, presented at IFSO 2019, Madrid, Spain

How long does the balloon stay in?

After approximately

16_{weeks}

the balloon's time-activated release valve will automatically deflate the balloon. It then passes out when you use the toilet^{1*}

1. Jamal et al. Obes Surg. 2019 Apr;29(4):1236-1241

In one study


45%

of patients did not notice the balloon passing¹


*In rare cases, the Elipse Balloon may require endoscopic or surgical intervention for removal. Talk to your doctor today to find out if the Elipse Programme is right for you. You can also learn more at www.allurion.com.



Weight Loss with Gastric Balloons

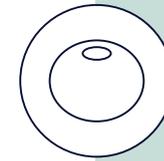

WEIGHT LOSS CURVE AFTER ORBERA IGB PLACEMENT¹

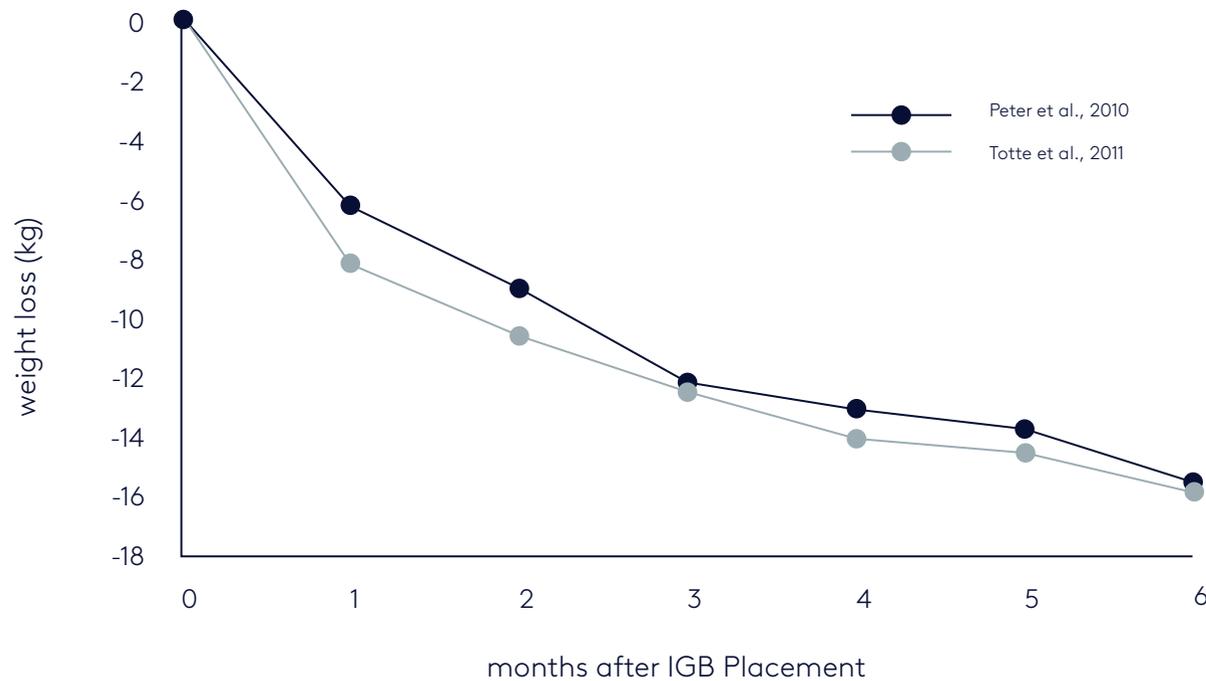

These are results from Orbera Gastric Balloon

80%

of weight loss of a 6-month Orbera balloon occurs in the first three months¹

1. Gaur S, et al. Gastrointestinal endoscopy, 2015 81: 1330-1336

Will I put the weight back on?

No weight-loss programme is a magic bullet



The Elipse Programme is designed to help you lose weight while you establish new lifestyle and eating habits*

*Results may vary from one participant to another.
A small percentage of patients don't respond to the treatment
**The Elipse Balloon must be used in conjunction with a supervised nutrition and behaviour modification programme. Allurion advises a 6-month follow-up programme. Its duration is defined by the physician, and may vary depending on the physician and patient profile.



As part of the programme, you will work with your doctor and nutritionist to devise a lifestyle plan to increase the chances of keeping the weight off**

ACHIEVE REAL RESULTS

26


“My experience with the Elipse Programme was great, and I kept losing weight after the balloon was gone”

Vanessa, Belgium
went from BMI 33.6 to BMI 28.5


Are there any expected side effects?

Common side effects generally subside within a week¹

- Symptoms may be controlled with medications prescribed by your physician.
- Most people are able to return to normal activities within a few days
- A clinical study of 8,506 Orbera patients showed the most common side effects were²

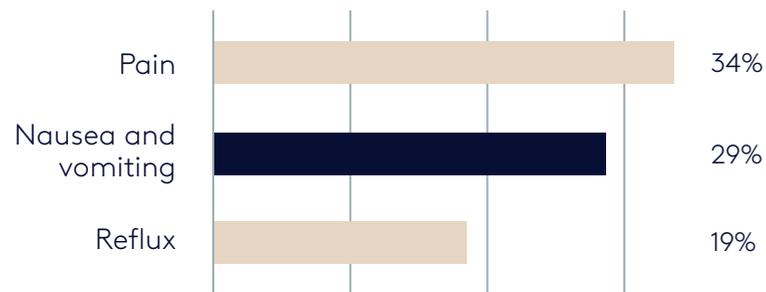


1. Raftopoulos and Giannakou. SOARD. 2017 Jul;13(7):1174-1182
2. Abu Dayyeh BK. Gastrointestinal Endoscopy. 2015;82(3):425-438

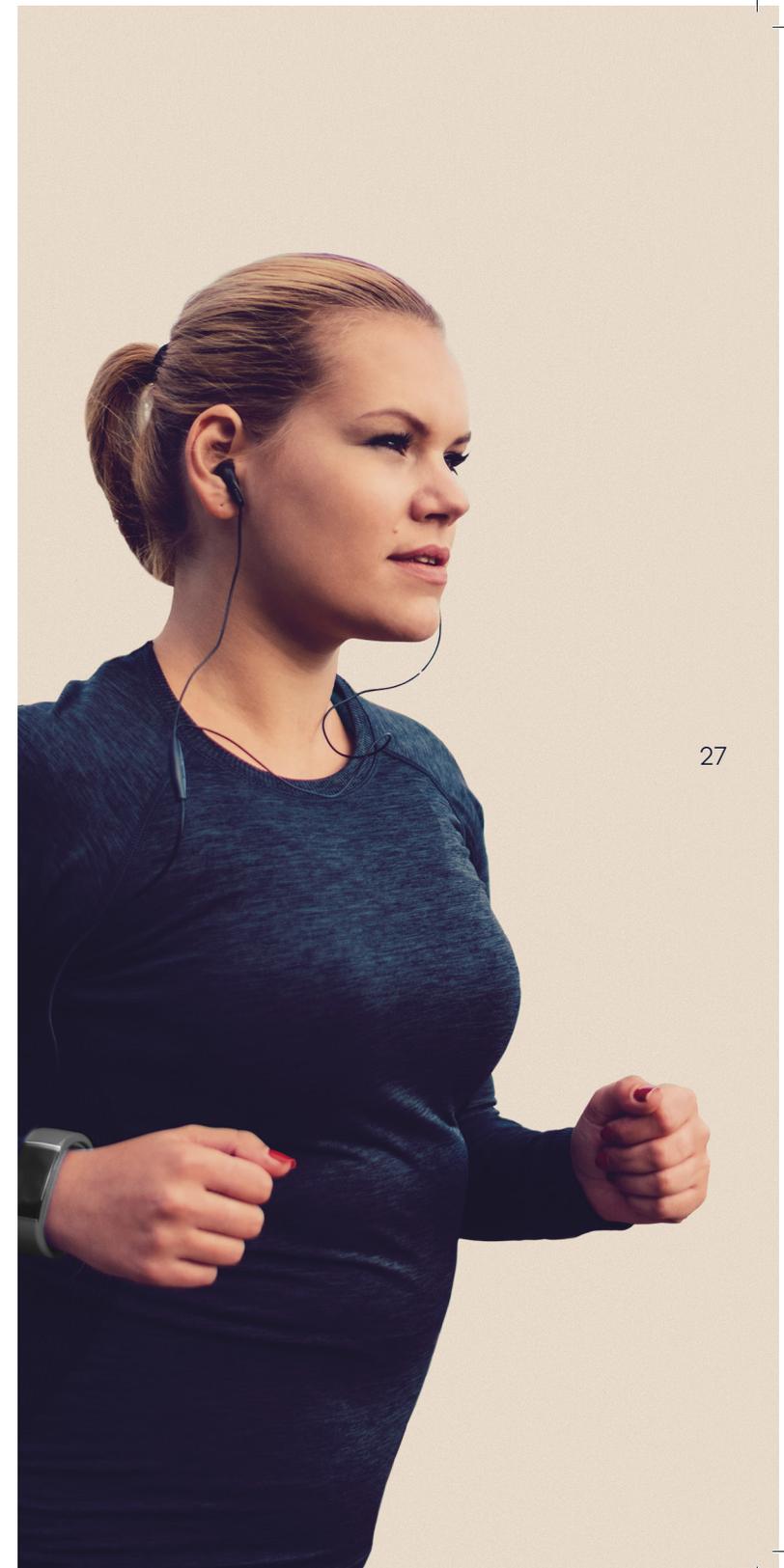

Making the most of the Elipse Programme



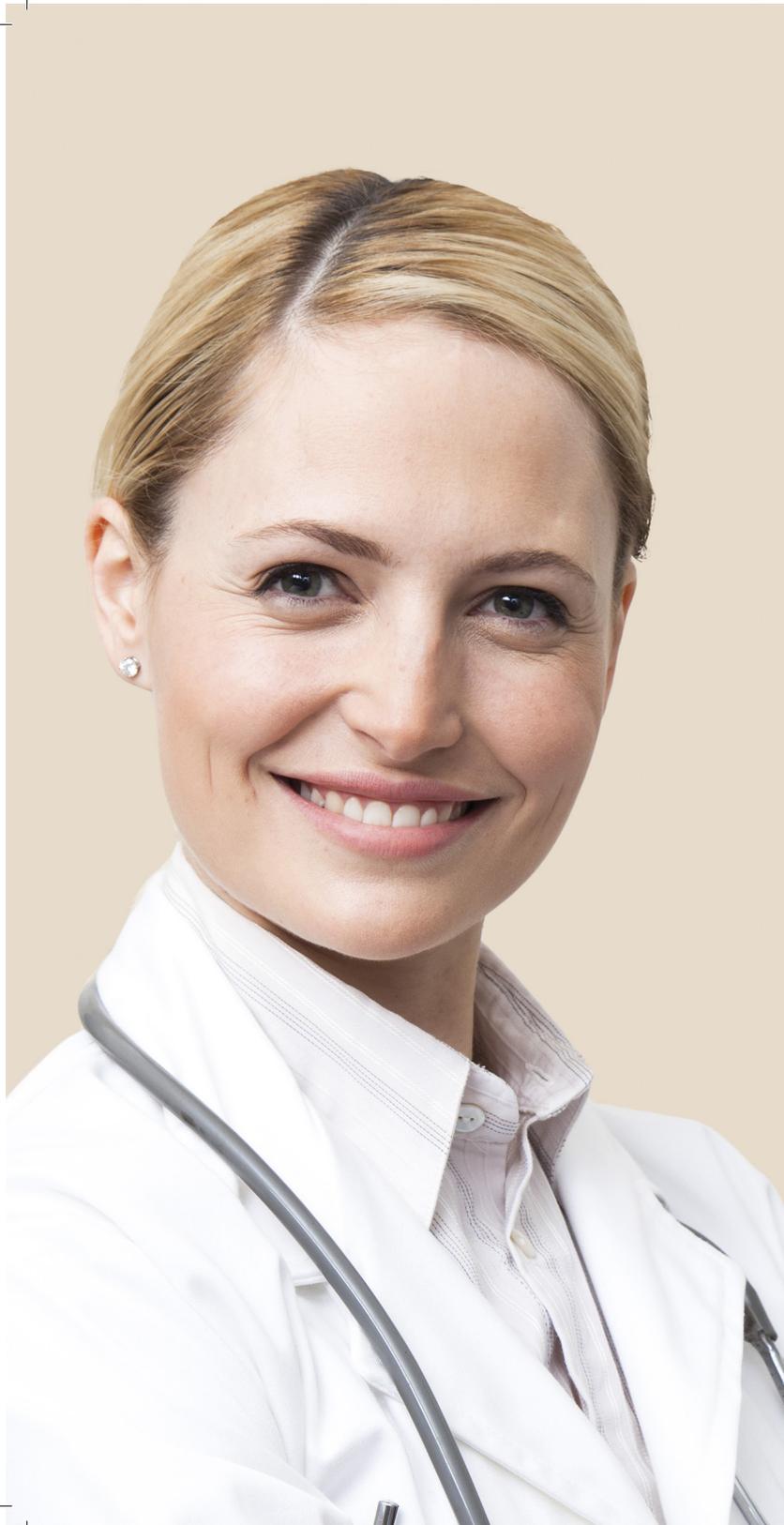

Before

Making the most of the Elipse Programme



Your doctor will guide you through the process but here are some tips.

- Some people experience side effects after their placement. Common side effects generally subside within a week¹
- It can be a good idea to book a few days off work afterwards just in case, or have the balloon placed on a Friday so you have the weekend off
- Ask a friend or family member to be on hand in case you need help with activities around the house




During


Making the most of the Elipse Programme



Follow your new healthy eating plan

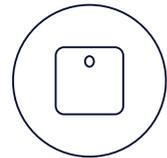

Use your Elipse scale and app to monitor your progress

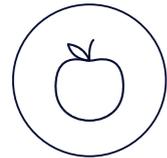

Work closely with your nutritionist and doctor*



Try and establish healthier lifestyle habits, such as exercising regularly and reducing your alcohol intake



Ask for support from friends and family to help you eat well and stay active

*The Elipse Balloon must be used in conjunction with a supervised nutrition and behaviour modification programme. Allurion advises a 6-month follow-up programme. Its duration is defined by the physician, and may vary depending on the physician and patient's profile.

After

Making the most of the Elipse Programme



Your weight-loss journey continues after your balloon has left your body

- Maintain your healthy eating plan
- Continue using your Elipse scale and app to monitor your progress
- Try and maintain healthier lifestyle habits, such as exercising regularly and reducing your alcohol intake
- Ask your healthcare professional if you need additional advice or support
- Remind friends and family that you will still need to eat well and stay active





Life changing

Vanessa, 38, Belgium

-13kg / 29lbs*

- Elipse Balloon placement date: Jan 2017
- From obese at BMI 33.6 to BMI 28.5

“Being a Policewoman, physical fitness is important to me. My experience with the Elipse Programme was great, and I kept losing weight after the balloon was gone*”

Actual Elipse Patient

*Results may vary from one participant to another. A small percentage of patients don't respond to the treatment.

Testimonials

Inspire others: share your patient success stories

“Before, I was so big that I had a lot of difficulties with mobility and there were days when I wouldn't be able to leave the house. Now, I feel like I have my life back again”

Michael, 67, UK, in the Daily Mail

“I swallowed the capsule without too much difficulty – I felt a bit queasy but nothing more (...) I've lost 25 inches all over and I can fit into a size 16 comfortably”

Kayleigh, 28, UK, in the Daily Mail – Spire Thames Valley Dr Humadi

“It was fantastic (...) I've lost 16 pounds in 4 weeks, my energy and my confidence are back, I'm fitting into clothes I haven't worn for 3 years!”

Hetsie, 42, UK, on Channel 4

“I was going straight to obesity (...) I finished the treatment 6 months ago but I still feel full faster than before. (...) I lost 20 kilos thanks to the Elipse Balloon”

Véronique, 50, France, in Femme Actuelle

Actual Elipse Patient

*Results may vary from one participant to another. A small percentage of patients don't respond to the treatment.



Five points to remember



01

Your doctor will decide if the Elipse Programme is right for you

1. Jamal et al. Obes Surg. 2019 Apr; 29(4): 1236-1241
2. Raftopoulos and Giannakou. SOARD. 2017 Jul;13(7):1174-1182.
3. Al Sabah, S., et al. SOARD. 2018, 14(3), 311-317.
4. Machytka E et al. Endoscopy. 2017



02

Gastric Balloons are not a magic bullet and require sustained lifestyle changes

5. Genco et al. Obes Surg 2017,DOI 10.1007/s11695-017-2877-1
6. Al-Subaie S et al. Int J Surg. 2017 Dec; 48:16-22.
7. Ienca et al. Submitted to SOARD.



03

After approximately 16 weeks of Elipse therapy, patients lose an average of 10 to 15kg (1.57 to 2.36 stone) or around 10 to 15% of their total bodyweight^{1 to 10}

- Abstract A278, presented at Obesity Week, 2018, Nashville, USA
8. Genco et al. Submitted to SOARD. Abstract A185, presented at Obesity Week, 2018, Nashville, USA
9. Ienca et al. Abstract 237, presented at IFSO 2019, Madrid, Spain
10. Raftopoulos et al. Abstract 1978. Presented at ACS 2019, San Francisco, United States.



04

Get multidisciplinary support during your weight loss journey*

*The Elipse Balloon must be used in conjunction with a supervised nutrition and behaviour modification programme. Allurion advises a 6-month follow-up programme. Its duration is defined by the physician, and may vary depending on the physician and patient's profile.



05

Discover a breakthrough in simplicity: no surgery, endoscopy or anaesthesia **

**In rare cases, the Elipse Balloon may require endoscopic or surgical intervention for removal.

Start your weight loss journey, Join the Elipse Programme



This medical device is a regulated healthcare product, which under European regulations bears the CE mark. The Elipse Balloon is a temporary weight loss aid for adults over the age of 18 who are overweight or obese, defined as a body mass index (BMI) ≥ 27 . The Elipse Balloon can only be administered by a healthcare professional who is knowledgeable about the use of the Elipse Balloon. The Elipse Balloon must be used in conjunction with a supervised nutrition and behaviour modification programme. Some individuals may not be eligible to receive the Elipse Balloon. Your doctor will take a detailed medical history and perform a physical examination to determine your eligibility. It is important to provide your physician with a complete and accurate medical history. This content is intended for audiences outside of the US.